

Trionic Walker Backrest - Assembly Instruction



The ergonomic and padded backrest, made from water-repellent and durable 600D polyester, fits the rollators Walker 9er, 12er and 14er. It offers a comfortable and safe support when you are seated.

The length/depth of the backrest is adjustable.

1 Unscrew.

Remove the screws and nuts from the two clamps.

2 Attach the backrest.

Open the clamps a little and attach them to the grip tubes. IMPORTANT: Make sure that the "TRIONIC" logotype is positioned correctly, as the backrest is angled.







4 fasten the backrest.

With a 4 mm allen key, fasten both clamps with the screws and nuts. The straps of the backrest should point forward at the point of attachment.

5 This photo shows the correct assembly.





Before unfolding the rollator.

To unfold the rollator, angle the backrest upwards.

If necessary.

If you find the backrest being too short/too long, you can adjust the length. Remove the two screws and nuts and then assemble the backrest again.

